



**CLICK**  
MEDICAL

**QuickFit™ Lacer**  
by CLICK

OK0240-230-05  
Size: Large





For help and video instructions:  
[clickmedical.co/quickfit-lacer](http://clickmedical.co/quickfit-lacer)

#### Install Hints:

Use QuickFit™ Tool 1/8" punch to make required holes.

If using existing holes for rivet guides, remove metal or plastic grommets from eyelets to ensure a secure rivet setting.

Rivets can be hand snapped to position for verifying fit, but system must be set with QuickFit™ Tool.

Press rivets at 90° with QuickFit™ Tool; pressing an angle will result in poor rivet setting.

#### Recommended Tools:



- QuickFit™ Setter Tool
- QuickFit™ Rivet Pack
- Marker Pen




#### Advisory

Must be installed by a trained professional.  
Lace will wear. At minimum, inspect monthly.  
Overtightening could reduce circulation.  
Do not use on patients with nerve issues.  
Keep lace away from sharp edges.


See advisories in other languages: [clickmedical.co/advisories](http://clickmedical.co/advisories)

 <p><b>6 MONTHS WARRANTY</b></p>	<p><b>WARRANTY</b> US customers contact Click Medical directly: Help@ClickMedical.co Help Line: +1.970.670.7012</p> <p>International customers, please contact your local distributor</p>	<p><b>MDSS GmbH</b> Schiffgraben 41 30175 Hannover, Germany</p> <p><b>Click Medical, LLC</b> 1205 Hilltop Parkway, W101 Steamboat Springs, CO, 80487, USA +1-970-670-7012</p>
		

#### Installation Guidelines:

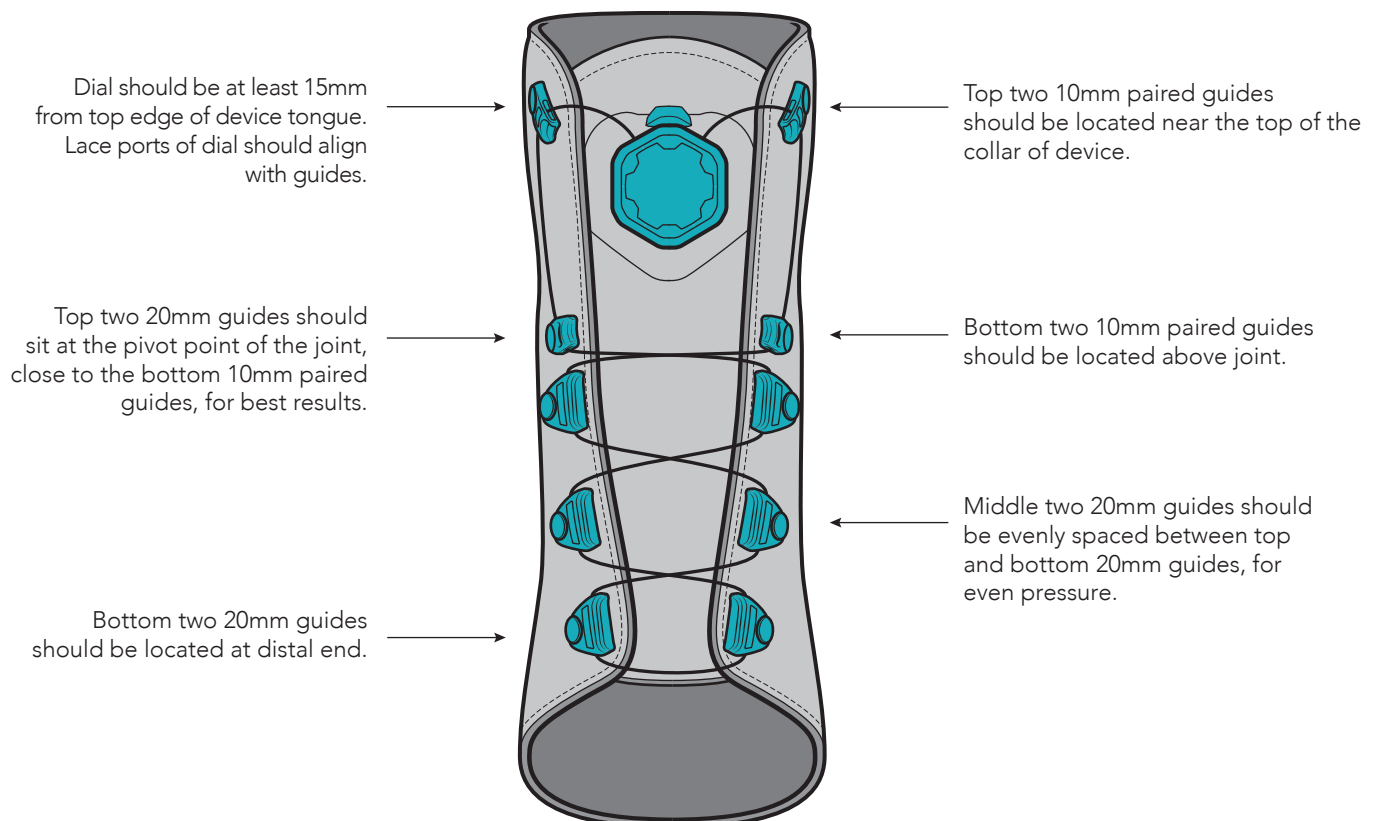


- 1 Determine guide and dial locations using installation guidelines. Punch holes using 1/8" punch option of QuickFit™ Tool.

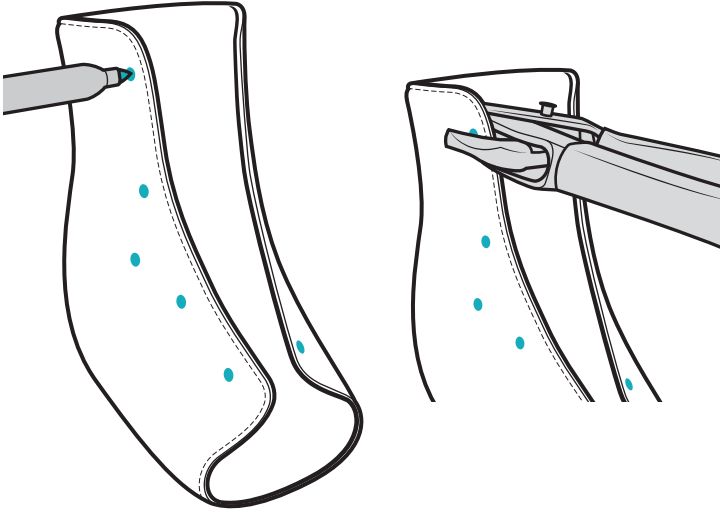


- 2 Use sticker to position system on device:


- a) If tongue is not removable, peel sticker from its backing paper and use it to position lacer on device.  
b) If the tongue is removable, keep sticker in contact with its backing paper and hand-set top two guides.

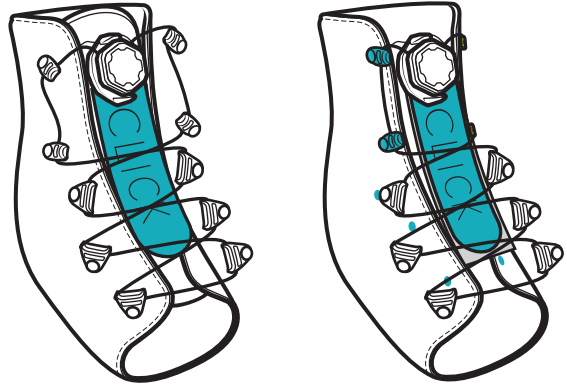


- 3 a) Attach lace guides by inserting male rivet from inside the device and through lace guide.  
b) Snap female rivet cap on top of lace guide, holding the guide to the shoe.

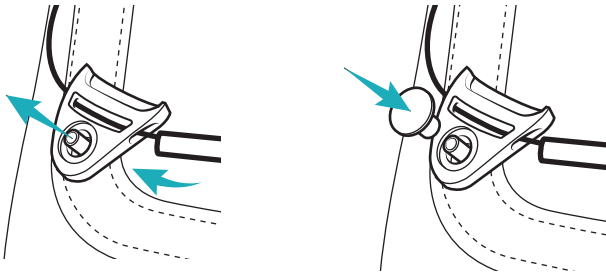

- 4 If satisfied with all placement, secure rivets using QuickFit™ Tool.

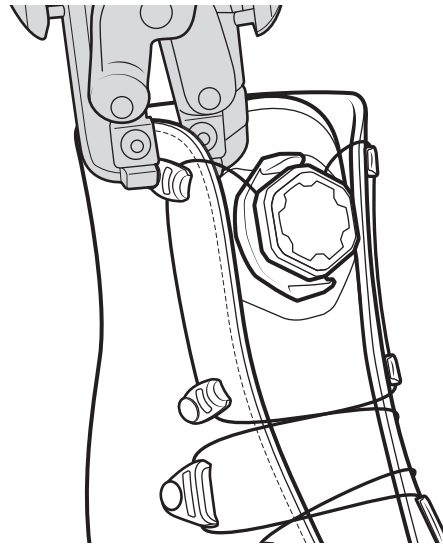

- 5 Rotate guides to correct lace alignment (see dashed line).

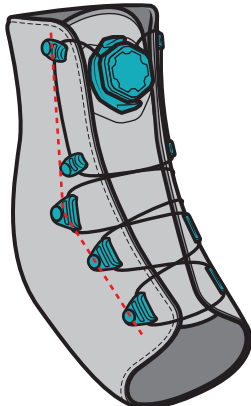

- 6 Cycle system to test function:  
Push to engage.  
Twist to tighten.  
Pull to release.

